

# Urgostérile®



## NON-WOVEN STERILE DRESSING

Indication: Covering superficial wounds and surgical wounds

- Non-adherent pad
- Soft and ventilated backing
- Hypoallergenic
- Extensible

## Urgostérile®

Supplied in boxes of individually pouched and sterile dressings, ready to use

SIZES	PAD SIZES	UNITS PER BOX
5.3 x 7 cm	2.9 x 3.5 cm	50 pouches
10 x 7 cm	7 x 3.5 cm	50 pouches
10 x 9 cm	7 x 4.5 cm	50 pouches
15 x 9 cm	11.5 x 4.5 cm	20 pouches
20 x 9 cm	15.8 x 4.5 cm	20 pouches
25 x 9 cm	19.4 x 4.5 cm	20 pouches
30 x 9 cm	23 x 4.5 cm	20 pouches
35 x 9 cm	26.75 x 4.5 cm	20 pouches

LABORATOIRES URGO ■ 42, RUE DE LONGVIC ■ 21300 CHENÔVE ■ FRANCE ■ TEL 00 33 3 80 54 50 00 ■ FAX 00 33 3 80 44 26 27  
[www.urgo.com](http://www.urgo.com)

**URGO**

# Urgostérile®



## DESCRIPTION

- Sterile dressing.

**COMPOSITION:** Urgosterile® is a dressing made of a non-woven, extensible polyester backing, coated with an acrylic adhesive mass and a central pad covered by a non-adherent polyethylene mesh.

**METHOD OF STERILIZATION:** Sterilized by ethylene oxide.

## PROPERTIES

- **Urgosterile®** is extensible: it follows the movements of the articulations or skin wrinkles, without tearing.
- Non-woven: micro-aerated, it prevents from maceration.
- Its acrylic adhesive mass provides a high skin tolerance (respect of sensitive skins).
- **Urgosterile®** does not stick to the wound thanks to its pad covered by a non-adherent polyethylene mesh.
- Long lasting adhesiveness with easy removal and full protection of the wound.
- **Urgosterile®** is sterile, thus reducing the risk of contamination.

## INDICATIONS

- **Urgosterile®** is indicated for an efficient and aseptic protection of the puncture points: parental feeding, biopsy, and dialysis.
- It is also indicated as a dressing for ambulatory surgery, post-surgery and coeliosurgery.

## DIRECTIONS FOR USE

### METHOD OF USE

- Cleanse the wound with saline solution.
- If an antiseptic is first used, rinse the wound thoroughly with saline solution before applying **Urgosterile®**.
- Apply **Urgosterile®** directly onto the wound. Avoid tensions when applying **Urgosterile®**.
- Check and change regularly.

### PRECAUTIONS

- If clinical signs of local infection appear, the treatment may be continued under medical supervision.
- **Urgosterile®** should be stored away from light, heat and moisture.
- Do not use if sterile protection has been damaged.

## LEGAL STATUS

CE 0459 Medical device